







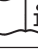





3. Mix the Reagent 1-Reagent 2 combined bottle thoroughly before each use, because colloidal gold particles may precipitate during storage. Mix it by slowly inverting to avoid creating bubbles. If bubbles occur, the instrument sensor will read an incorrect volume and the bottle will be deleted from the system. Pipette out any bubbles down to the level of the liquid.
4. As with all assays, the results of this test can be influenced by factors present in some patients' specimens.
5. For diagnostic purposes, the results obtained from this assay should always be used in combination with a clinical examination, patient medical history, and other findings.
6. Procedural directions must be followed exactly because any modification of the procedure may change the results.
7. Use of reagents, disposables, or spare parts other than those supplied by the authorized distributor may produce incorrect results.
8. Store the reagents according to the storage methods. Do not use them after the expiration date.
9. Use fresh feces.
10. This test should not be used to analyze specimens taken from a patient who is menstruating or who has hemorrhoids.
11. The test has not been validated for testing of patients with hemoglobinopathies.

17.0 LITERATURE REFERENCES

1. Yasuhiro Oono et al., A retrospective study of immunochemical fecal occult blood testing for colorectal cancer detection, *Clinica Chimica Acta*, 411, p802–805, 2010.
2. Edward Randell et al., Evaluation of Hemo Techt NS-Plus system for use in a province-wide colorectal cancer screening program, *Clinical Biochemistry*, 46(4–5), p365–368, 2013.
3. Setsuko Kato et al., Basic Evaluation of Hemotect NS-Prime, an Automated Immunochemical Analyzer for Fecal Occult Blood Testing, *J Clin Lab Inst Reag*, 37(3), p371–377, 2014.
4. Wael L.L.Demian et al., Evaluation of the analytical performance of the novel NS-Prime system and examination of temperature stability of fecal transferrin compared with fecal hemoglobin as biomarkers in a colon cancer screening program, *Practical Laboratory Medicine*, 2, p29–36, 2015.
5. Ari Ahn, M.D. et al., Performance Evaluation of Two Automated Quantitative Fecal Occult Blood Tests, *Lab Med Online*, 6(4), p233–239, 2016.
6. Taira Inagaki et al., Evaluation of the new automatic immunochemistry fecal occult blood analyzer "Hemo Techt NS-Prime", *Japanese Journal of Medical Technology*, 65(2), p222–228, 2016.
7. Takenobu Shimada et al., How should an adequate recall rate in colorectal cancer screening be evaluated and dealt with? , *Journal of Gastrointestinal Cancer Screening*, 55(1), p31–44, 2017.

18.0 SYMBOLS USED IN PRODUCT INSERTS AND ON LABELS

Symbols	Meanings of the symbols
	Expiry date (Used by...)
	Lot number
	Catalogue code
	Manufactured by
	Authorized EC representative
	Number of tests
	<i>In vitro</i> diagnostic medical device (<i>In vitro</i> diagnostic use)
	Temperature limitation (store at...)
	See instruction for use
	Reagent 1
	Reagent 2
	May damage fertility May damage unborn child

19.0 DATE OF ISSUE OR REVISION

October 1, 2022